Guide til spytprøvetagning

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Guide to Saliva Collection

Instructions for saliva collection

Please follow the instructions below when performing the saliva collection. The supervisor is responsible for ensuring that sampling is carried out in accordance with the guidelines provided.

IMPORTANT!

Consuming food and/or beverages as well as smoking within 30 minutes before sampling may affect the quality of the sample.

Produce saliva by placing the tongue against the root of the upper or lower jaw.

Use the funnel of the collector to collect **2 mL saliva** sample (air bubbles is not included).


Keep the **Collection Tube** upright, open the cap of the preservation medium tube. Save the cap for step 6.

Add all preservation medium into the **Saliva Collector Tube**.

Keep the collector upright and loosen the funnel of **Saliva Collector Tube**.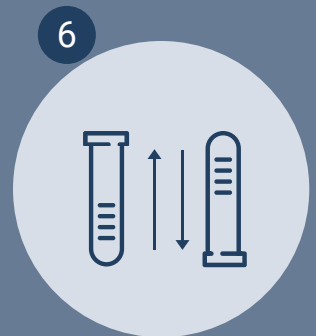

Close the cap of **Saliva Collector Tube** and mix the medium for a couple of seconds.

IMPORTANT!

1. If the preservative comes in contact with eyes or skin, rinse immediately with clean water and seek medical attention.
2. Do not ingest or swallow the cap on the collection tube, as this may cause suffocation.
3. If there is a liquid leak from a pipe, do not use the saliva collection, but discard it.
4. The Collection Tube (the funnel) and the empty liquid tube are thrown away as regular waste.