



Produce saliva by placing the tongue against the root of the upper or lower jaw.



Use the funnel of the collector to collect 2 mL saliva sample (air bubbles is not included).




Keep the Collection Tube upright, open the cap of the preservation medium tube. Save the cap for step 6.



Add all preservation medium into the Saliva Collector Tube.



Keep the collector upright and loosen the funnel of Saliva Collector Tube.

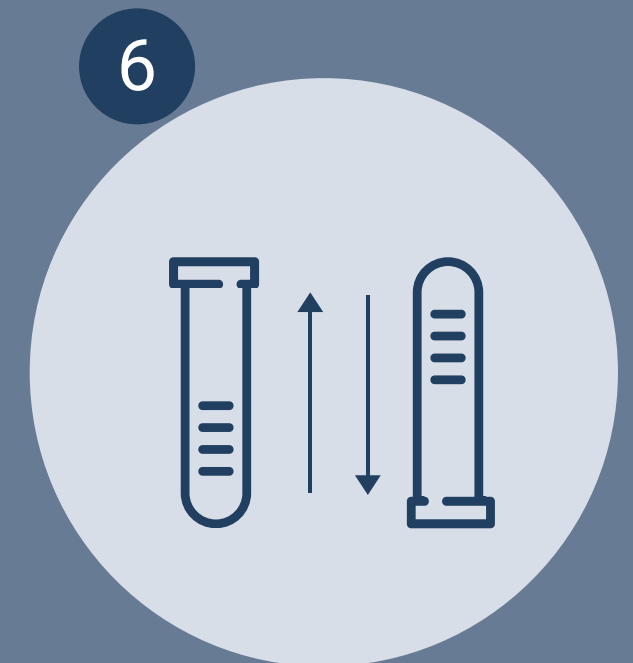

Close the cap of Saliva Collector Tube and mix the medium.

1



Du skal producere spyt ved at trykke spidsen af din tunge mod ganen og køre spidsen rundt.

2



Spyt forsigtigt i Opsamlingsrør, indtil den flydende spyt (uden luftbobler) når op til den blå linje (2 mL)

3



Hold Opsamlingsrøret lodret og skru låget af Konserveringsvæske. Gem låget til brug i sidste trin.

4




Hæld forsigtigt væsken i Opsamlingsrør og kassér herefter det andet rør.

5



Hold Opsamlingsrør lodret, skru tragten af røret og kassér tragten.

6

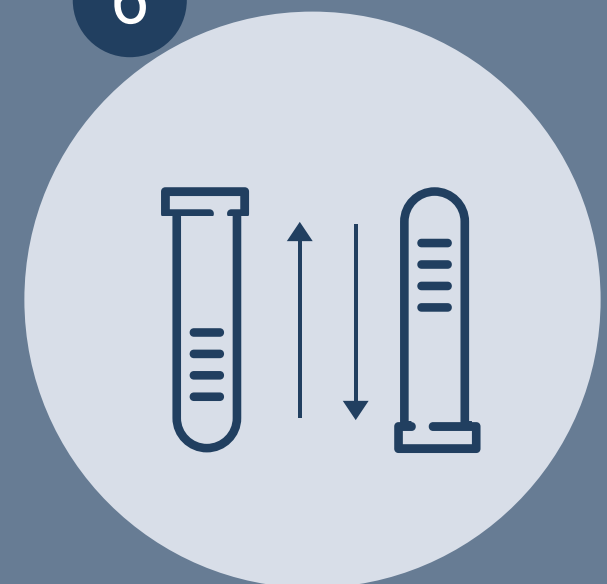

Skru låget fra Konserveringsvæsken fast på Opsamlingsrør og ryst røret.