

COVID-19 PCR - Guide til spytttest


STEP 5: Indlevering af prøve(r)

Når du har gennemført alle de foregående trin, og du har modtaget en bekræftelse på din bestilling, er du klar til at indlevere prøven hos dit valgte PentaBase-laboratorie.

- 1 Placer prøverøret i en zip-lockpose. Sørg for at alle rør er skruet tæt.



- 2 Luk zip-lockpose med prøverør i

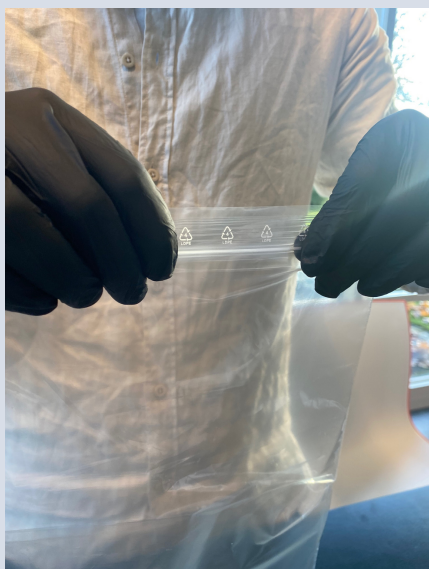

- 3 Udfyld den vedlagte delivery note og aflever denne sammen med prøven/prøverne.

- 4 Indlever zip-lockpose til jeres valgte PentaBase laboratorie.



COVID-19 PCR - Guide for test


STEP 5: Delivery of sample(s)

When you have completed the previous steps, and you have received a confirmation of your order, you are ready to deliver your samples to your chosen PentaBase laboratory.

- 1 Place your sample tube in a zip-lock bag. Make sure to close the lid on the sample tight.



- 2 Close your zip-lock bag with the samples.

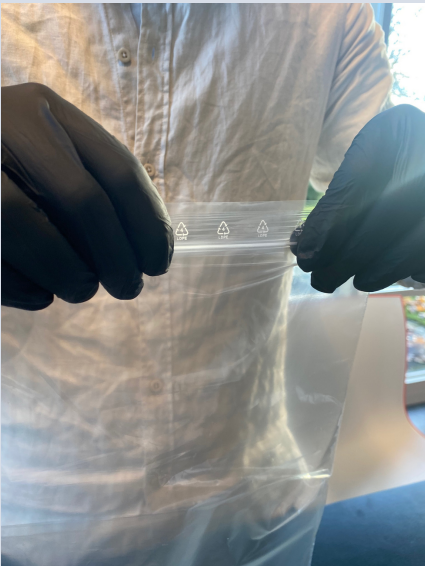

- 3 Fill out the attached delivery note and deliver this together with your sample(s).

- 4 Deliver your zip-lock bag to your chosen PentaBase laboratory.