

Guide til spytprøvetagning

Registrering i GDPR-valideret system

Forud for prøvetagningen skal medarbejderne oprette en profil i PentaBases journalsystem. Opret profilen her: <https://journal.pentabase.com>, og følg anvisningerne. Man kan oprette sig både på telefon og e-mail.

Oprettelse af prøve/prøvenummer

Før prøven foretages, skal prøvens unikke prøve-ID oprettes i systemet.

Følg venligst nedenstående guide:

- 1 Login på testpersonens profil på PentaBase Journal: <https://journal.pentabase.com>
Du kan både bruge din computer og din telefon.

OBS! Der skal altid '+landekode' foran dit tlf. nr. ved login. Ex. +45XXXXXXXX


- 2 Klik på **'Scan selv'** i menulinjen.



- 3 Nu skal prøverørets QR kode eller nummer registreres.

a. Scan QR koden med dit kamera på enten din computer eller telefon.

Hvis QR kode ikke kan scannes, indtastes prøvenummer / Sample ID ved at klikke "Enter Sample ID"
Det er vigtigt, at prøvenummeret står på præcis samme måde, som på røret.

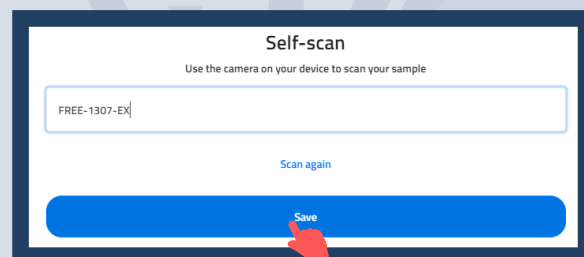

b. Klik **'Save'**. Du har nu registreret din prøve i PentaBases journalsystem.

Guide for Saliva Collection

Secure account in GDPR-validated system

Employees must register a profile in Pentabase's file system before the saliva collection. Register your profile here: <https://journal.pentabase.com>

Registration of sample/sample ID

You must register the unique sample-ID in the Pentabase system before the saliva collection. Please follow the instructions below;

- 1 The person getting tested must sign in to their profile on Pentabase Journal: <https://journal.pentabase.com>
You can both use your computer and your mobile phone.
NOTE: Always remember to choose '+country-code' before your mobile phone number when signing in.
Ex. +45XXXXXXXXX



- 2 Click on '**Self-Scan**' in the top menu.



- 4 Register the QR code or sample ID on the tube.

a. Scan the QR code by using the camera on either your computer or your mobile phone.

If the QR code cannot be scanned, you have to type in the sample number/sample ID by clicking on "Enter Sample ID".
It is important that the sample ID is typed in exactly as it is written on the tube.

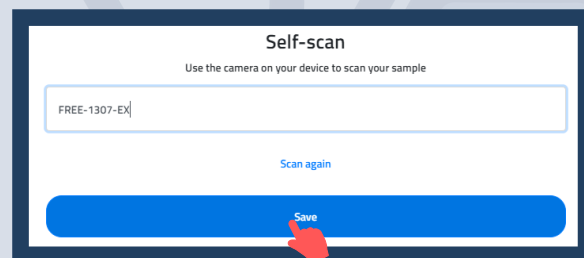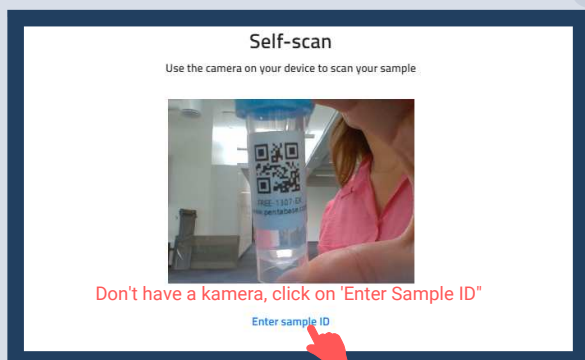

b. Click '**Save**'. You have now registered your sample in the PentaBase file system.